<b>Doküman No: IFU-GDPN</b> Documentation No: IFU-GDPN	<b>İlk yayın tarihi: 02/05/2012</b> Valid from: 02/05/2012	<b>Revizyon no: 01</b> Revision no: 01	<b>Sayfa:1</b> Page:1
		<b>Revizyon tarihi: 26.05.2016</b> Revision date:26.05.2016	

**GEOTEK® DİREKT GİRİŞ NEFROSTOMİ SETİ****Kullanma Kılavuzu****TR**

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**GEOTEK®DIRECT PUNCTURE SET****Instruction for Use****ENG**

Direct Puncture Nephrostomy set includes;Pigtail type drainage catheter with metal stiffener and trocar point stylet, bag connector. Nephrostomy catheter made by Tecoflex PU .

**Indications:** Temporary External Drainage of renal collection sistem obstructed by inflammation, renal tumors..etc

**Contraindications:** Serious infections, hematuria, incontinence

**Side Effects:** Kidney, upper bladder irritaiton caused by foreignbody, pain, infection, increased miction-frequency, hematuria

**Cautions:** This Kit should be used by only by physicians experienced in the use of such device.


Regular monitoring of location and free passage of catheter is necessary to evaluate nephrostomy efficiency.

Ethylene oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30 °C temperature, protect from sun light.. Possible allergic reactions should be considered. For single Patient Use Only. Do not attempt to clean or reesterilize this product.

**Placement of Nephrostomy Catheter:**

- Insert rounded tip metal stiffener in to catheter in order to straighten the pigtail
- Load the trocar point stylet in to metal stiffener and catheter.
- Close the luer lock mechanism by screwing hubs of catheter, stiffener and stylet.
- Insert the Direct puncture Nephrostomy catheter according to current medical standards.
- When the catheter placement is complete, unscrew and withdraw the metal stiffener and trocar stylet gently while holding the catheter at right position.
- Connect the catheter with urine bag with bag connector.

**Note:** These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.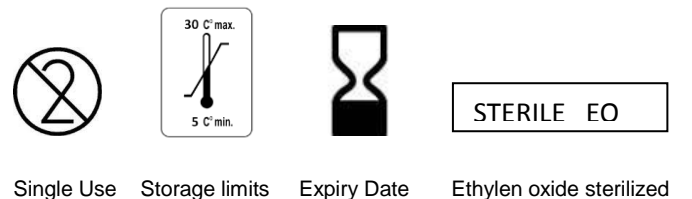

1984 : CE mark and NB identification number